



Contact Us

O / 855-722-6006

F / 310-755-3108


[service@qtmedical.com](mailto:service@qtmedical.com)

# QT Cloud USER'S MANUAL

<https://dashboard.qtmedical.com/login>



Web service version:  
V1.15.4 (Singapore)  
V1.15.5 (Other regions)

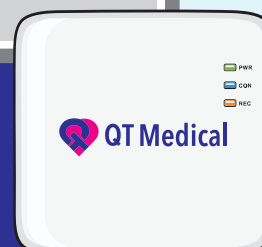

Version: V2-R7

Update Date: 04/01/2023

# Table of content

## QT Cloud (Web Service)

<b>1. Registration</b>	<b>2</b>
1.1 Registration Forms	2
1.2 Consent Forms (PCAdmin Account ONLY)	5
<b>2. Account Activation and Setting</b>	<b>7</b>
2.1 Account Activation	7
2.2 Account Information	8
2.3 Password	9
2.4 Notification	12
<b>3. Account Creation (PCAdmin account ONLY)</b>	<b>13</b>
3.1 Physician Creation	13
3.2 Operator Creation	16
<b>4. ECG Records</b>	<b>18</b>
4.1 Overview	18
<b>5. ECG Summary View</b>	<b>20</b>
5.1 Measurement Adjustment (Physician Account ONLY)	20
5.2 Caliper (Physician Account ONLY)	22
5.3 ECG Diagnosis (Physician Account ONLY)	24
<b>6. Patient Profile Management</b>	<b>26</b>
6.1 Overview	26
<b>7. Assign</b>	<b>29</b>
7.1 Patient Profile to ECG	29

## Customer Service Support

31



# QT Cloud (Web Service)

## 1. Registration


### 1.1. Registration Forms

A one-time setup is required before you can begin using the QT ECG system. Have your Bluetooth-enabled mobile device ready and follow the steps below:

Input the following URL in your browser (**we currently only support the "Google Chrome" browser**) to open the QT Medical's dashboard.


<https://dashboard.qtmedical.com/login>

Select **"Activate"**.

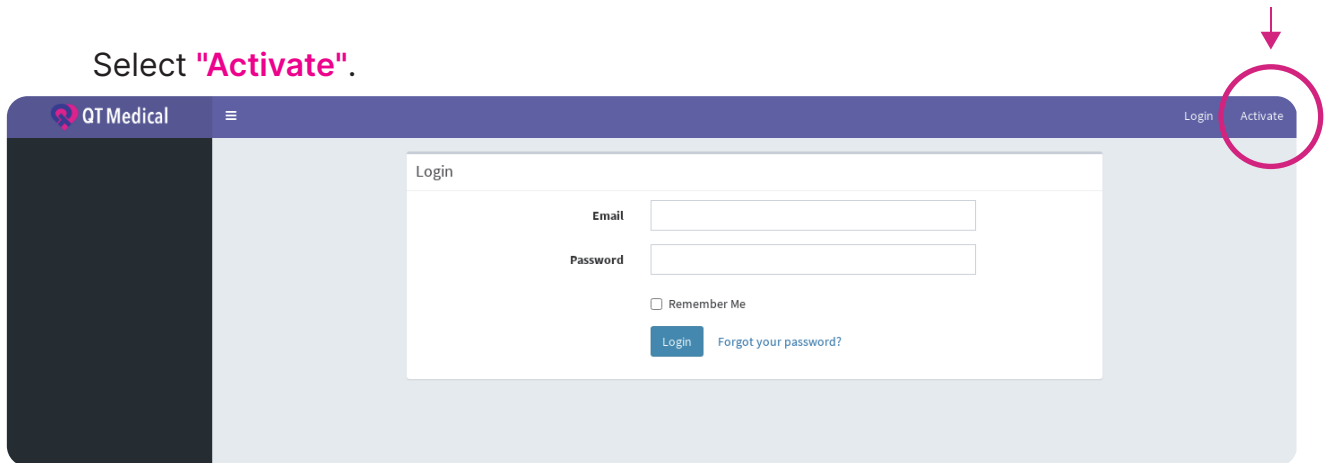


The screenshot shows the QT Medical dashboard. In the top right corner, there are two links: "Login" and "Activate". The "Activate" link is circled in red, and a red arrow points to it from above. The main content area displays a "Login" form with fields for "Email" and "Password", a "Remember Me" checkbox, and "Login" and "Forgot your password?" buttons.

Fill in the information, you can also create physician account at the same time by filling in your Physician Registration Number. Select **"Submit"** after when you complete the information.

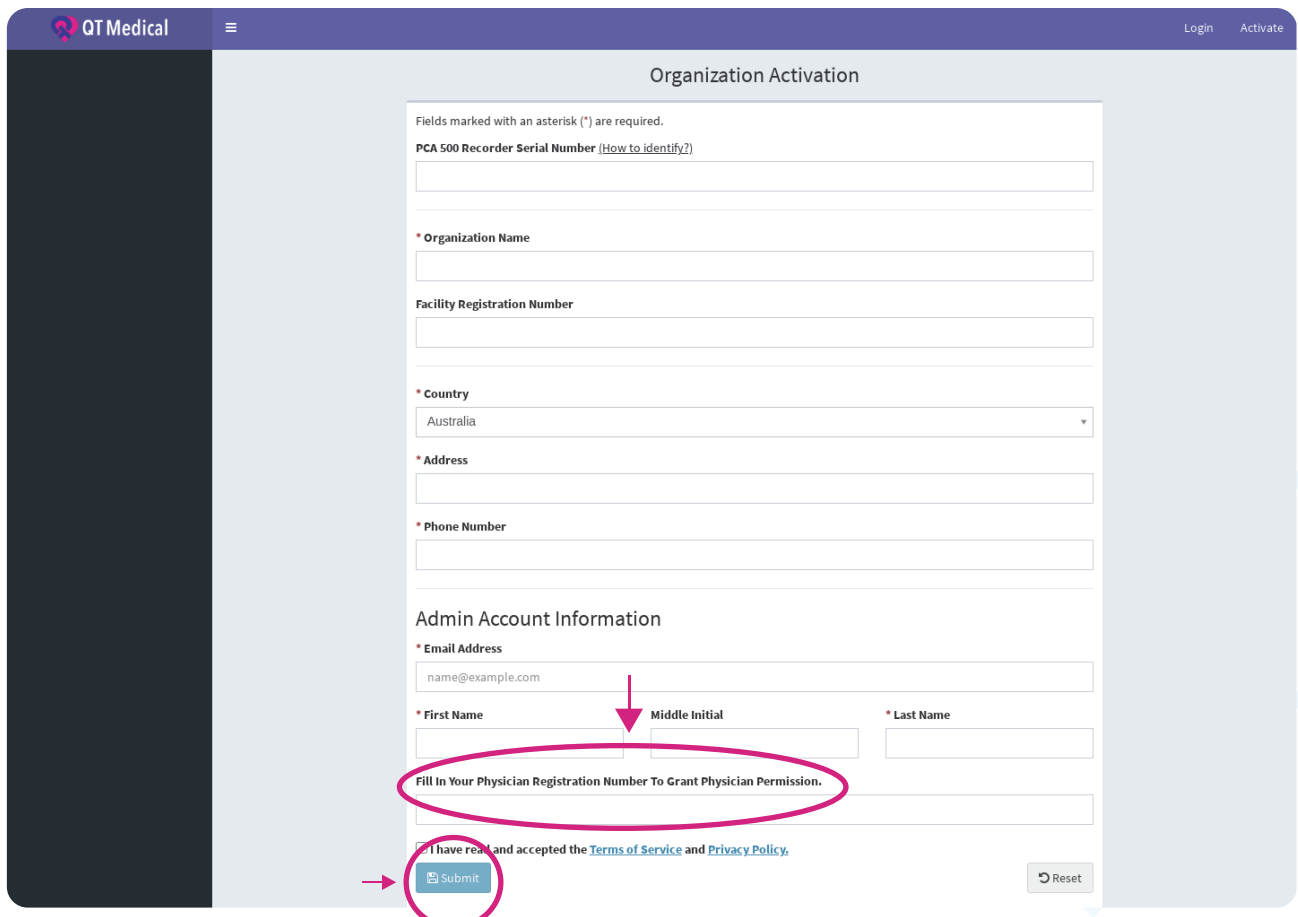


The screenshot shows the "Organization Activation" form. It contains several sections: "PCA 500 Recorder Serial Number" with a link to "How to identify?", "Organization Name", "Facility Registration Number", "Country" (a dropdown menu currently showing "Australia"), "Address", "Phone Number", "Admin Account Information" with fields for "Email Address" (pre-filled with "name@example.com"), "First Name", "Middle Initial", and "Last Name". Below these fields is a red oval containing the text "Fill In Your Physician Registration Number To Grant Physician Permission." and a text input field. At the bottom, there is a checkbox for "I have read and accepted the Terms of Service and Privacy Policy." and two buttons: "Submit" (circled in red with an arrow pointing to it from the left) and "Reset".

After submitting the information, you will need to **verify your email address**, follow the on-screen instructions.

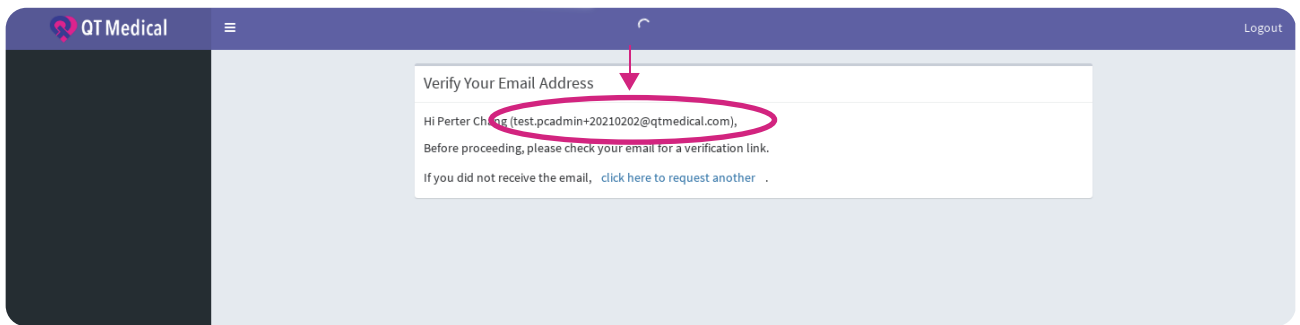


Check your email, and you will get your administrator password (please also check Spam Folder). Click the **"Verify Email Address"** to verify your email:


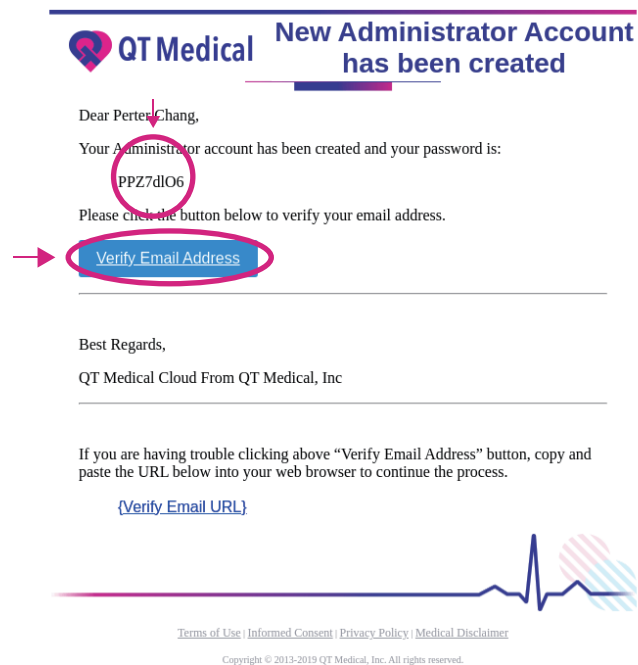




If you are having trouble clicking on the **“Verify Email Address”** button, copy and paste the URL below into your web browser to continue the process.

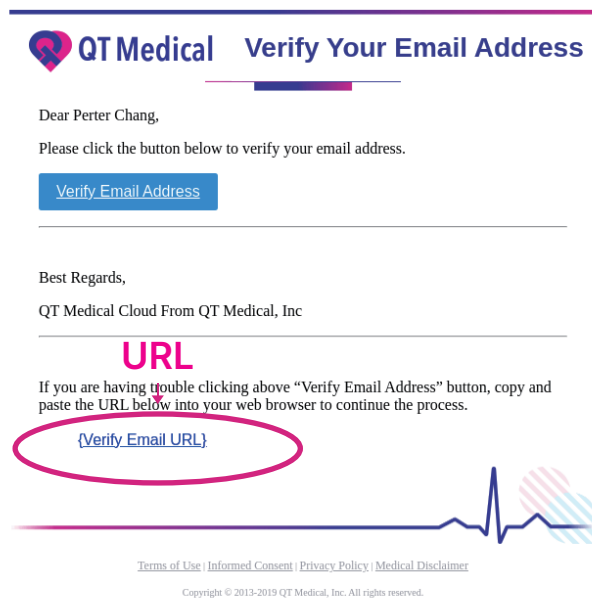


After successful verification, log into your account.

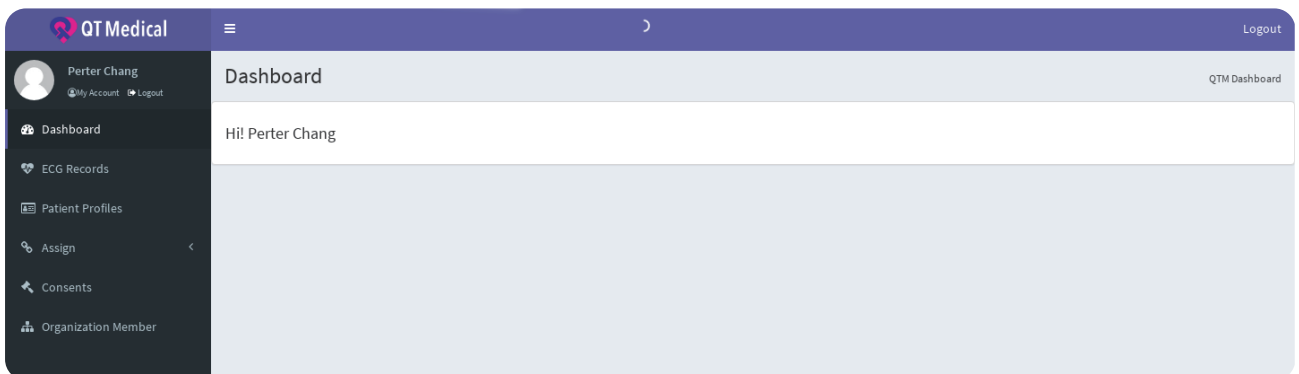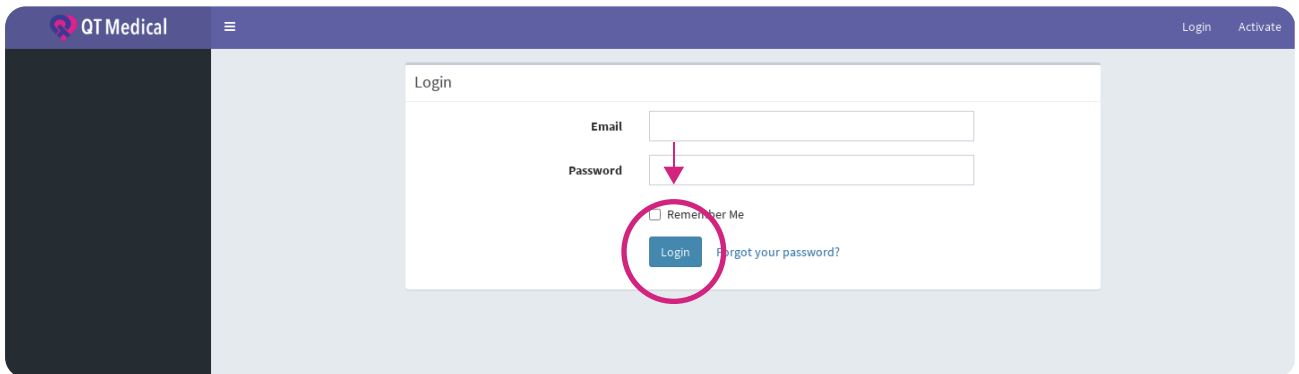


## 1.2. Consent Forms PCAdmin Account ONLY

When you log in for the first time, you will be prompted to agree to our consents.


Select the tab to read **"Terms of Use"**.

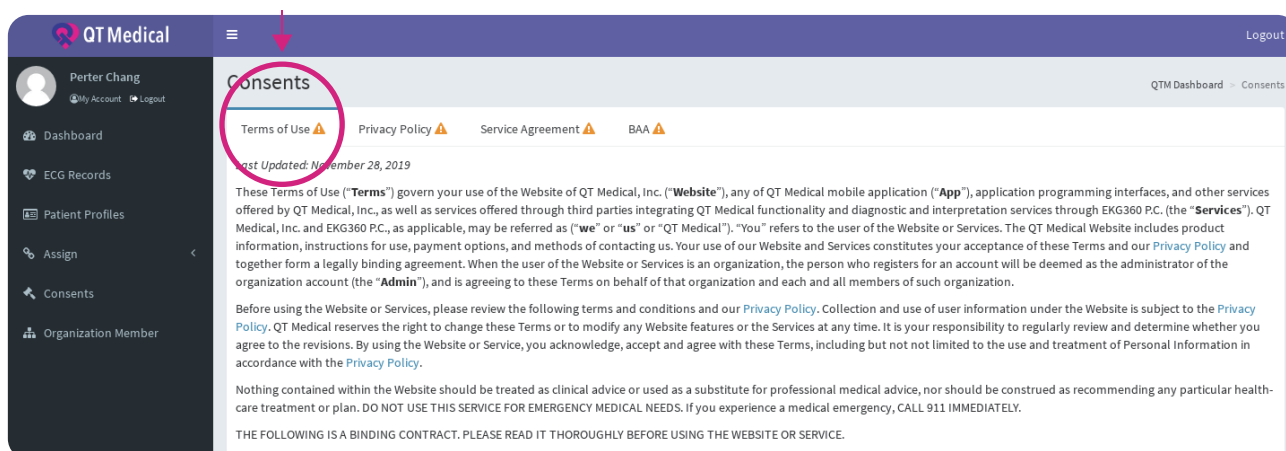

After reading the terms of use, tap **"Agree"**.

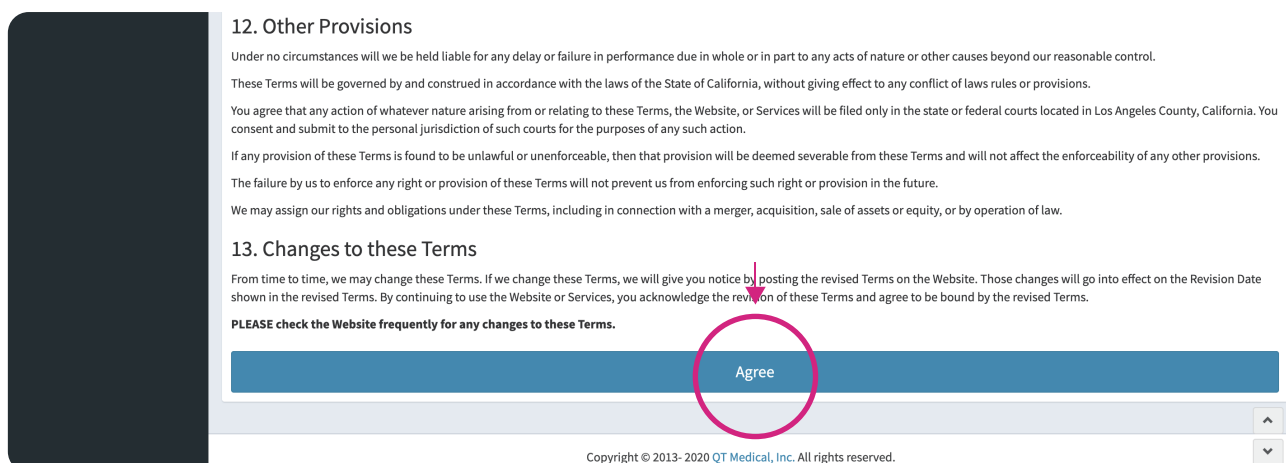

Select **"Privacy Policy"** to read.

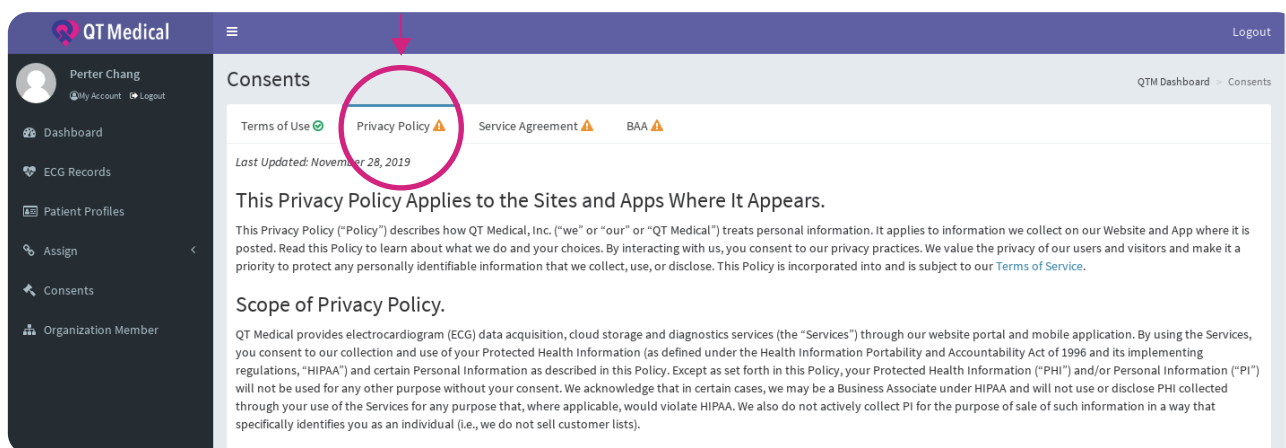

After reading the privacy policy, tap "Agree".

**You Can Contact Us About This Policy and Your Information.**

If you have any questions about this Privacy Policy or our data practices, please email us at [privacy@qtmedical.com](mailto:privacy@qtmedical.com). You can also write or call us at:

QT Medical, Inc.  
Attn: Brett Chien, PhD  
1360 Valley Vista Dr., Suite 203, Diamond Bar, CA 91765  
(909)323-0007

We will not take action against you for filing a complaint. If you have a complaint concerning our compliance with applicable privacy laws, we will investigate your complaint and take appropriate measures. You may also file a complaint relating to our use and disclosure of your PHI to the United States Department of Health and Human Services Office for Civil Rights at <https://ocrportal.hhs.gov/ocr/smartscreen/main.jsf>.

**We May Update This Policy.**

This Policy may be updated from time to time without further notice to you and this will be reflected by a "Last modified" date above. Please revisit this webpage regularly for any changes. By continuing to use the Services, you are consenting to the terms of the then-current Policy.

**Agree**

Copyright © 2013- 2020 QT Medical, Inc. All rights reserved.

After agreeing to all consents, you can see the summary view after logging in successfully.

**QT Medical** Logout

Perter Chang  
My Account Logout

Dashboard  
ECG Records  
Patient Profiles  
Assign  
Consents  
Organization Member

**Consents** QTM Dashboard > Consents

Terms of Use Privacy Policy Service Agreement BAA

Last Updated: November 28, 2019

These Terms of Use ("Terms") govern your use of the Website of QT Medical, Inc. ("Website"), any of QT Medical mobile application ("App"), application programming interfaces, and other services offered by QT Medical, Inc., as well as services offered through third parties integrating QT Medical functionality and diagnostic and interpretation services through EKG360 P.C. (the "Services"). QT Medical, Inc. and EKG360 P.C., as applicable, may be referred as ("we" or "us" or "QT Medical"). "You" refers to the user of the Website or Services. The QT Medical Website includes product information, instructions for use, payment options, and methods of contacting us. Your use of our Website and Services constitutes your acceptance of these Terms and our Privacy Policy and together form a legally binding agreement. When the user of the Website or Services is an organization, the person who registers for an account will be deemed as the administrator of the organization account (the "Admin"), and is agreeing to these Terms on behalf of that organization and each and all members of such organization.

Before using the Website or Services, please review the following terms and conditions and our Privacy Policy. Collection and use of user information under the Website is subject to the Privacy Policy. QT Medical reserves the right to change these Terms or to modify any Website features or the Services at any time. It is your responsibility to regularly review and determine whether you agree to the revisions. By using the Website or Service, you acknowledge, accept and agree with these Terms, including but not limited to the use and treatment of Personal Information in accordance with the Privacy Policy.

Nothing contained within the Website should be treated as clinical advice or used as a substitute for professional medical advice, nor should be construed as recommending any particular health-care treatment or plan. DO NOT USE THIS SERVICE FOR EMERGENCY MEDICAL NEEDS. If you experience a medical emergency, CALL 911 IMMEDIATELY.

THE FOLLOWING IS A BINDING CONTRACT. PLEASE READ IT THOROUGHLY BEFORE USING THE WEBSITE OR SERVICE.

**QT Medical** Logout

Perter Chang  
My Account Logout

Dashboard  
ECG Records  
Patient Profiles  
Assign  
Consents

**Dashboard** QTM Dashboard


Hi! Perter Chang




## 2. Account Activation and Setting

### 2.1. Account Activation

Check your email, and you will get your password. Click the **"Verify Email Address"** to verify your email:

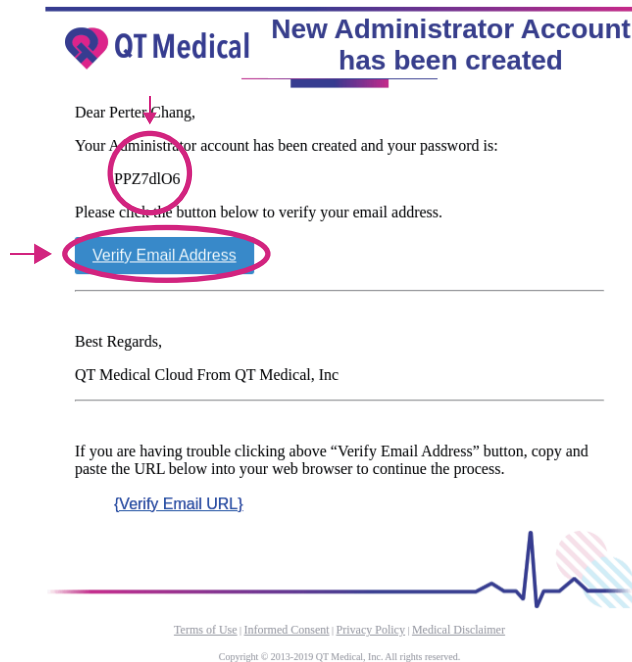


After successful verification, log into your account.

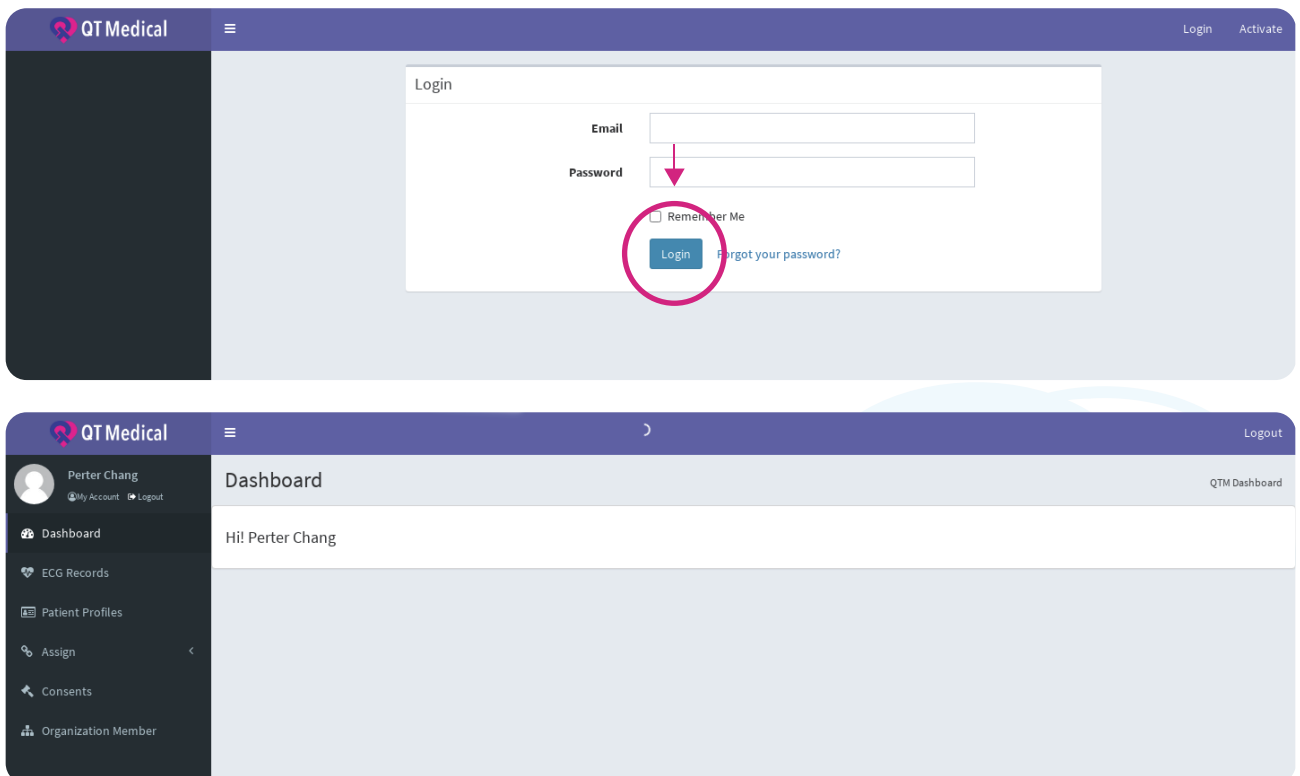


## 2.2. Account Information

### 2.2.1 Update the account info.

Click **"My account"**.

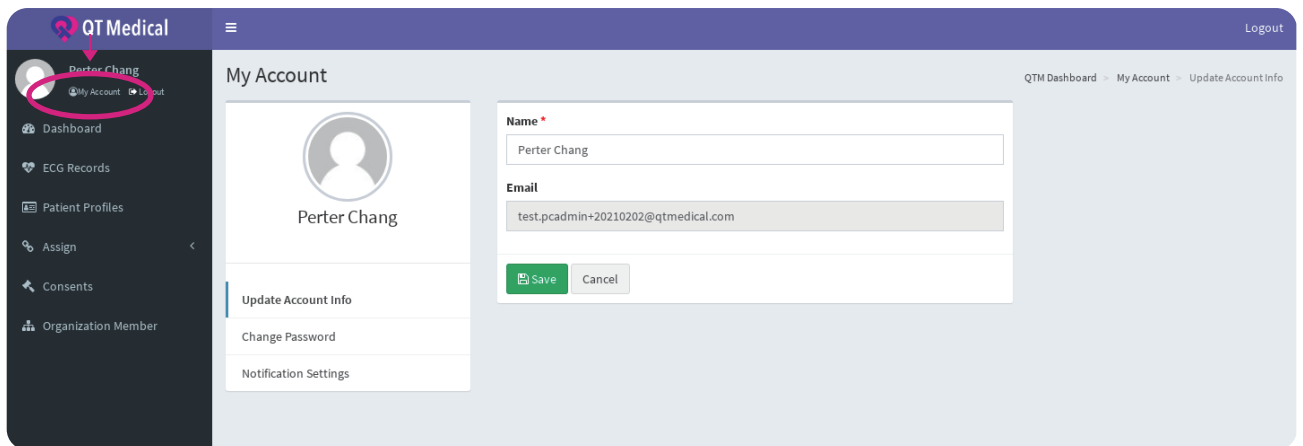


To change your account info, please select **"Update Account Info"**.  
Edit your info, click the Save button, you will see the **"Success"** popup .

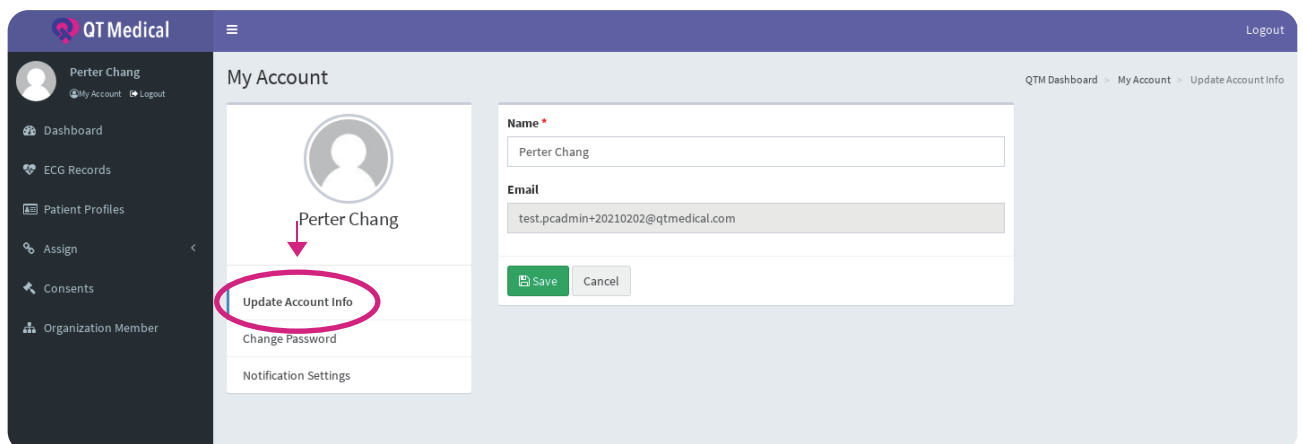

## 2.3. Password


### 2.3.1 Forget password

Click the **"Forgot your Password?"** .

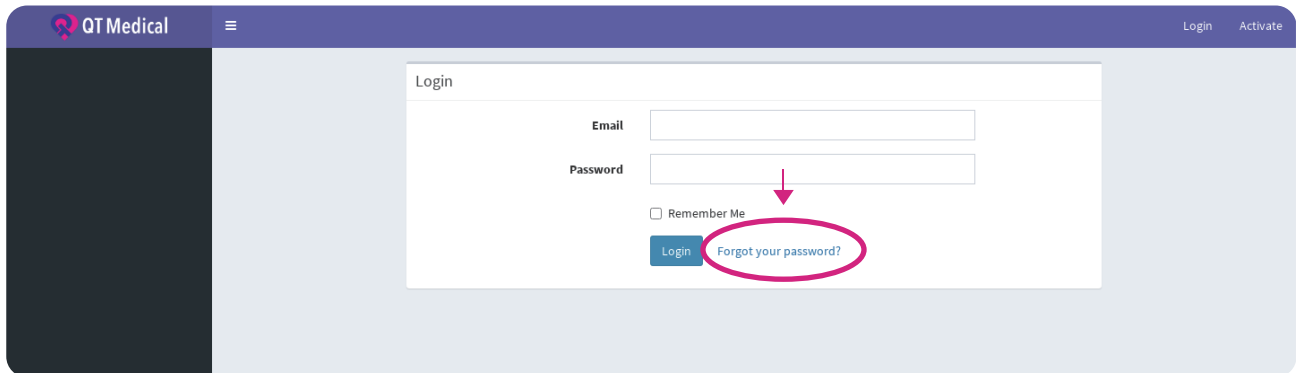


The screenshot shows the QT Medical login interface. At the top, there's a header with the QT Medical logo and a menu icon. Below the header, there's a login form with fields for Email and Password. A checkbox labeled 'Remember Me' is present. Below the password field, there's a 'Login' button and a link labeled 'Forgot your password?'. The link is circled in pink, and a red arrow points to it from the text above.

Enter your Email address and click the **"Reset Password"**.

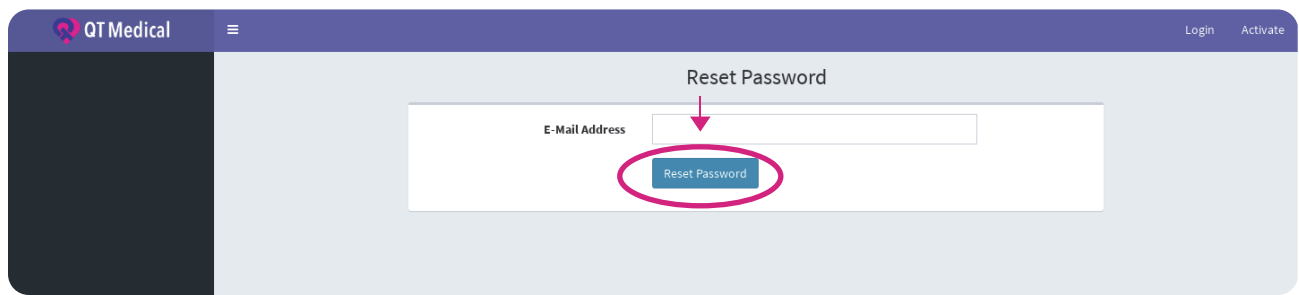


The screenshot shows the QT Medical 'Reset Password' page. It has a header with the QT Medical logo and a menu icon. Below the header, there's a form with a label 'Reset Password' and an 'E-Mail Address' field. Below the field, there's a 'Reset Password' button. The button is circled in pink, and a red arrow points to it from the text above.

Check your email, you will get the **"reset password notification"**.

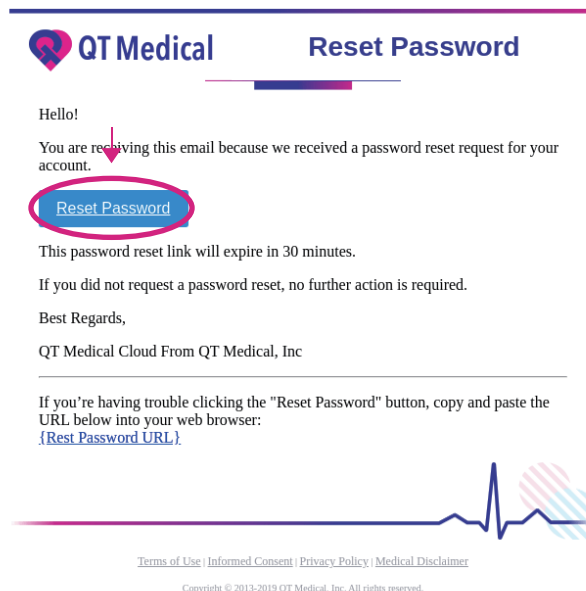

The screenshot shows an email notification from QT Medical. The header includes the QT Medical logo and the title 'Reset Password'. The body of the email starts with 'Hello!' and a red arrow pointing to the text 'You are receiving this email because we received a password reset request for your account.' Below this, there's a 'Reset Password' button circled in pink. The email continues with 'This password reset link will expire in 30 minutes.', 'If you did not request a password reset, no further action is required.', 'Best Regards,', and 'QT Medical Cloud From QT Medical, Inc'. At the bottom, there's a note about copying and pasting the URL if the button doesn't work, followed by a placeholder '{Rest Password URL}'. The footer contains links for 'Terms of Use', 'Informed Consent', 'Privacy Policy', and 'Medical Disclaimer', along with a copyright notice for 2013-2019 QT Medical, Inc.

Enter your Email and new password and click **"Reset Password"**.  
You will see the "Success" message popup .

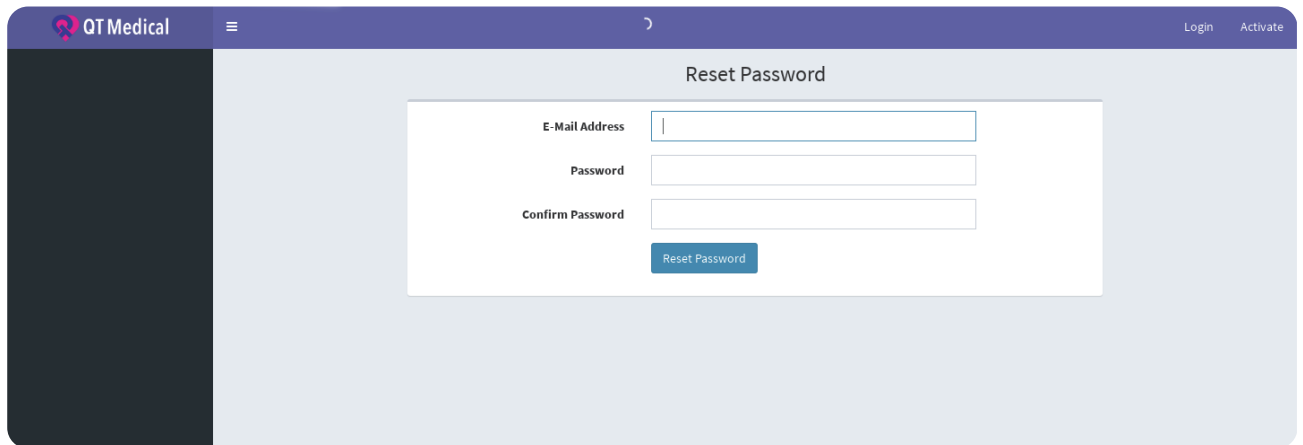

The screenshot shows a web application interface for "QT Medical". The header bar is dark blue with the "QT Medical" logo on the left, a hamburger menu icon, and "Login" and "Activate" links on the right. The main content area has a light blue background and is titled "Reset Password". It contains a white form with three input fields: "E-Mail Address", "Password", and "Confirm Password". Below these fields is a blue button labeled "Reset Password". A dark blue sidebar is visible on the left side of the page.



## 2.3.2 Change password

Select **"Change Password"** to set your new password.

QT Medical

Jon Snow  
My Account Logout

Dashboard  
ECG Records

My Account

Update Account Info  
**Change Password**  
Notification Settings

Old Password \*

Old Password

New Password \*

New Password

Confirm Password \*

Confirm Password

Change Password Cancel

QTM Dashboard > My Account > Change Password

After setting your new password, please select **"Change Password"**.

QT Medical

Jon Snow  
My Account Logout

Dashboard  
ECG Records

My Account

Update Account Info  
**Change Password**  
Notification Settings

Old Password \*

Old Password

New Password \*

New Password

Confirm Password \*

Confirm Password

Change Password Cancel

QTM Dashboard > My Account > Change Password





## 2.4. Notification Settings

Click the "**Notification Settings**" and you can select the options below.

Physician Account ONLY : Gets immediate notification when ECG Uploaded.

QT Medical

Perter Chang  
My Account Logout

Dashboard  
ECG Records  
Patient Profiles  
Assign  
Consents  
Organization Member

Notification Settings

Update Account Info  
Change Password  
Notification Settings

For Operators

- ☒ When an ECG is uploaded
- ☒ When an ECG Report is ready

For Physicians

- ☒ QT Cloud ECG Daily Summary
- ☐ QT Cloud ECG Hourly Summary
- ☐ When New ECG Recording is uploaded

Save Cancel

You will receive your notifications.

QT Medical ECG Recording is uploaded

Dear Jon Snow:

Your ECG recording is uploaded and your physician has been notified.

Sequence Number	Patient Name	Upload Time
1	*****	2021/01/26 10:01:55 PM PST

ECG recording has been sent to QTMedical Cloud Services for your physician to read. You will receive an email notification when the report is ready.

If you need any assistance, please contact QTMedical support team at: [service@qtmmedical.com](mailto:service@qtmmedical.com)


[Terms of Use](#) | [Informed Consent](#) | [Privacy Policy](#) | [Medical Disclaimer](#)

Copyright © 2013-2019 QT Medical, Inc. All rights reserved.

### 3. Accounts Creation PCAdmin Account ONLY

#### 3.1 Physician Account Creation

Please select **"Organization Member"** to create and edit accounts for your organization.

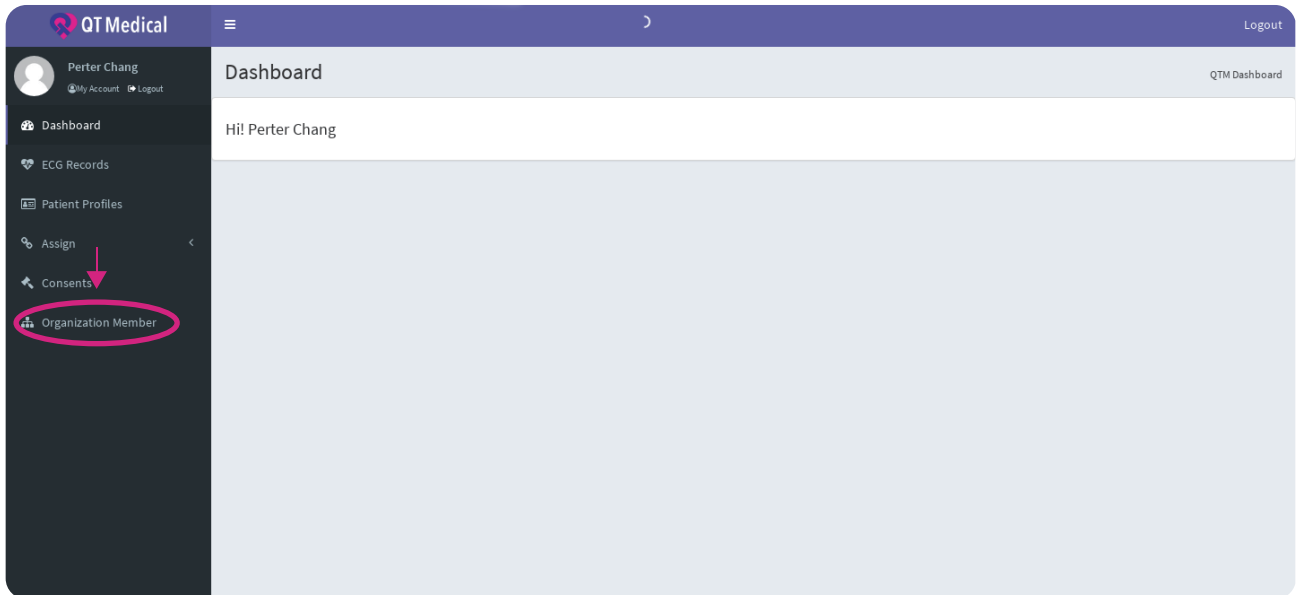


Fill in the information, you can create two types of accounts, physician or operator, and select **"Submit"** after when you complete the information.


The screenshot shows the 'Organization Member' page with two main sections: 'New Physician Account' and 'New Operator Account'. Both sections have a 'Quota Usage' indicator (3/100 for physician, 5/100 for operator). The 'New Physician Account' section includes fields for First Name, Middle Init, Last Name, Email Address, and Physician Registration Number. The 'New Operator Account' section includes fields for First Name, Middle Init, Last Name, and Email Address. Both sections have a 'Submit' button and a 'Reset' button. A red circle highlights the 'Submit' button in the 'New Operator Account' section, with a red arrow pointing to it and the text 'Create new operator.' next to it. Another red circle highlights the 'Submit' button in the 'New Physician Account' section, with a red arrow pointing to it and the text 'Create new physician.' next to it. Below these sections, there are lists for 'Physician Account (1/100)' and 'Operator Account (1/100)'.

After submitting the information, the message will be shown below. Inform the email's owner to verify the email address and follow the on-screen instructions.


Check the physician's email for the password (please also check Spam Folder). Click the Verify Email Address to verify the email:

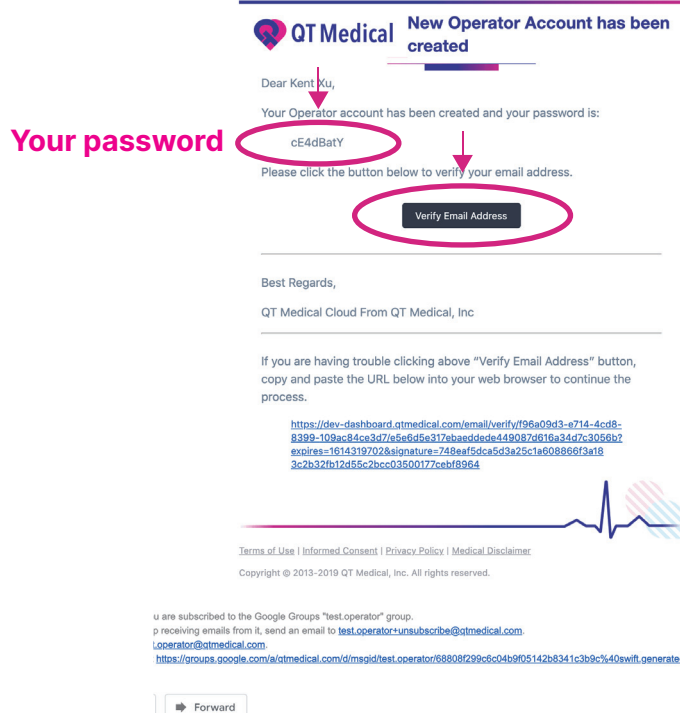

If you are having trouble clicking on the **“Verify Email Address”** button, copy and paste the URL below into your web browser to continue the process.

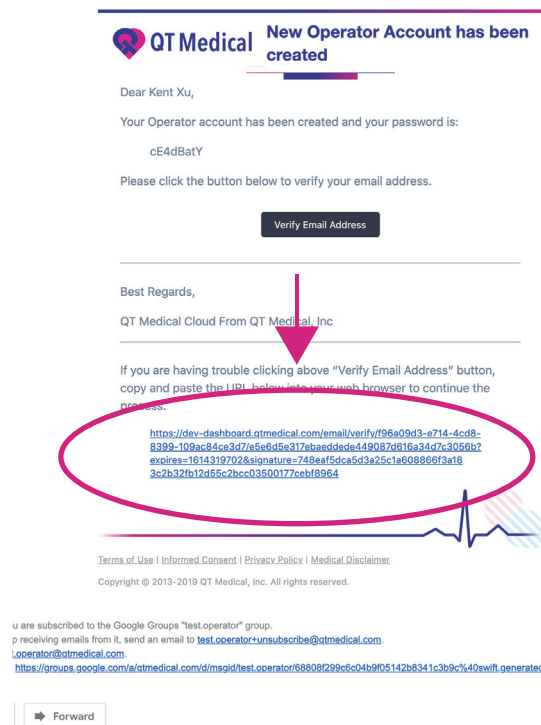

After verifying the email address, you will see the summary view.

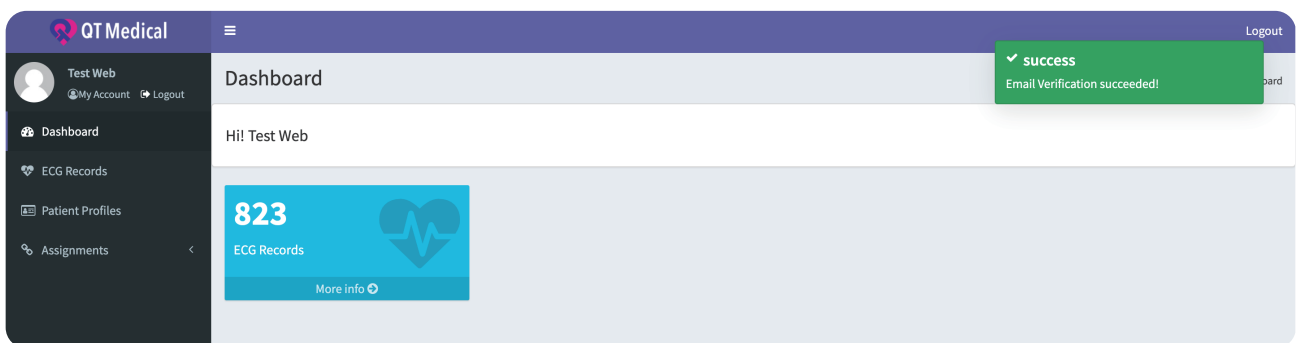

## 3.2 Operator account Creation


Fill in the information, you can create two types of accounts, and select **"Submit"** after when you complete the information.

The screenshot shows the 'Organization Member' page in the QT Medical system. The left sidebar contains navigation links: Dashboard, ECG Records, Patient Profiles, Assignments, Consents, and Organization Member. The main content area is divided into two columns. The left column contains the 'New Physician Account' form, which includes fields for First Name, Middle Init, Last Name, Email Address (test.pcadmin@qtmedical.com), Physician Registration Number, and License info / Practice state. The 'Submit' button is circled in pink with the text 'Create new physician.' The right column contains the 'New Operator Account' form, which includes fields for First Name, Middle Init, Last Name, and Email Address (name@example.com). The 'Submit' button is circled in pink with the text 'Create new operator.'


After submitting the information, the message will be shown below. Inform the email's owner to verify the email address and follow the on-screen instructions.

The screenshot shows the 'Organization Member' page after account creation. A green success message 'success Create Operator account success.' is displayed at the top right. The 'New Operator Account' form is now filled with: First Name: Kent, Middle Init: Xu, Last Name: Xu, and Email Address: test.operator+202101027@qtmedical.com. The email address field is circled in pink. The 'New Physician Account' form is also filled with: First Name: Test, Middle Init: Web, Last Name: Web, and Email Address: yt.pai+20210127@qtmedical.com. The 'Email Address (Not Verified)' label is visible above the email address field.


Check operator's email for the password (please also check Spam Folder). Click the Verify Email Address to verify the email:

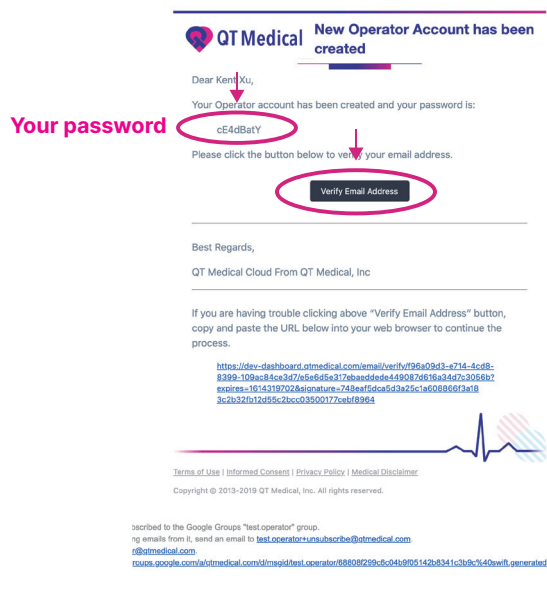

If you are having trouble clicking on the **"Verify Email Address"** button, copy and paste the URL below into your web browser to continue the process.

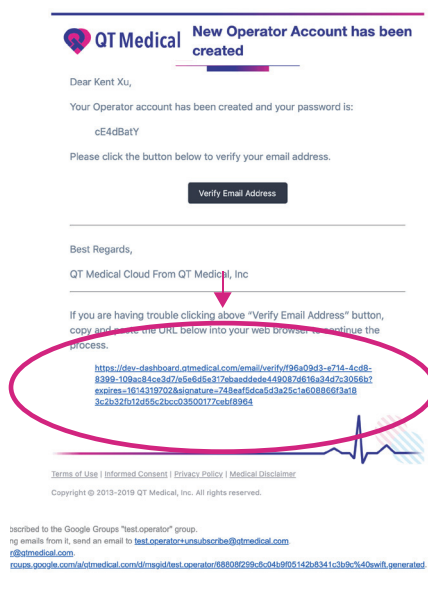


After verifying the email address, you will see the summary view.

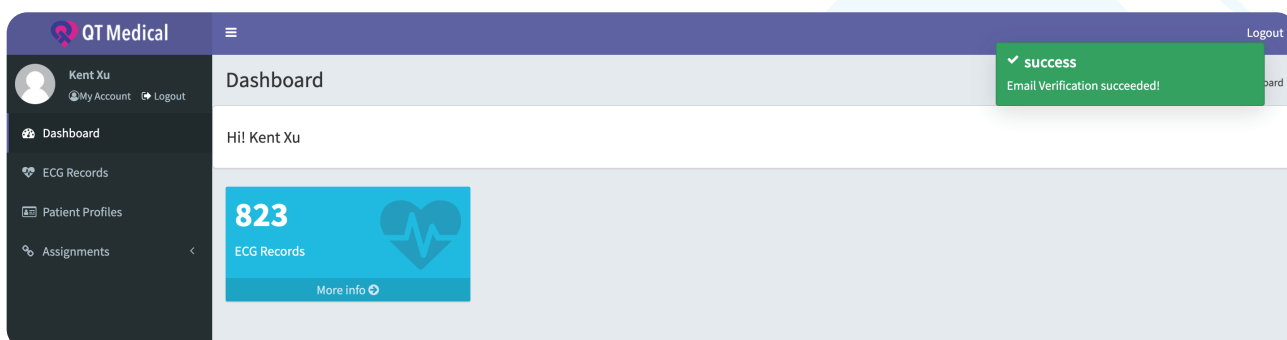


## 4. ECG Records

### 4.1 Overview

4.1.1 Tab the **"ECG Records"**, you can see all uploaded ECG records from the dashboard.

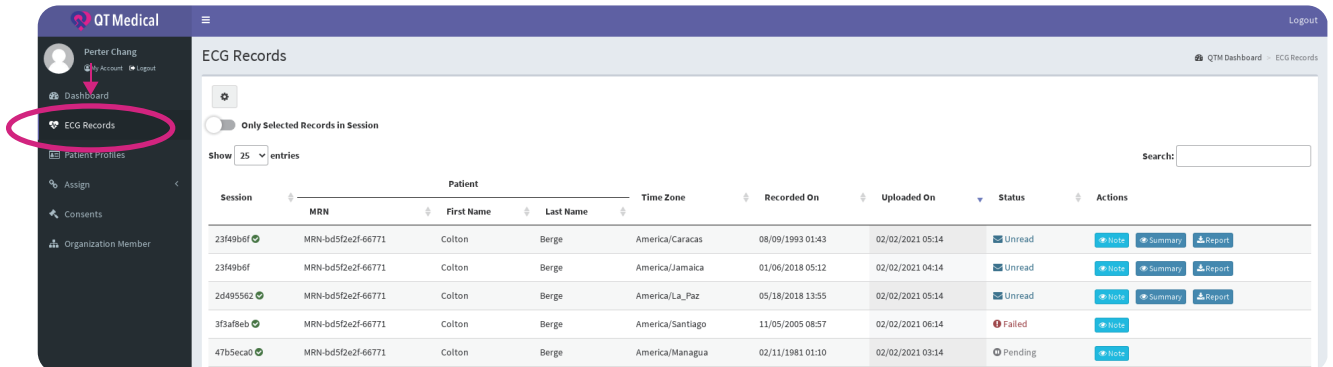


ECG Records recorded during the same session will be grouped as shown below.

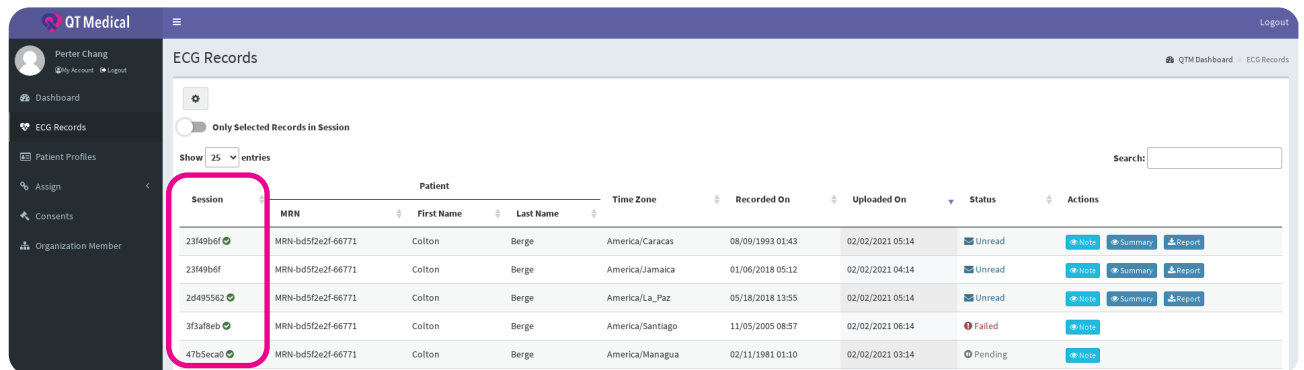

#### 4.1.2 Status

- **Unread:** Your report hasn't been diagnosed.
- **Finalized:** Your report have been diagnosed.
- **Failed:** If status shows "Failed", please contact QT Medical customer service.

<https://www.qtmedical.com/contact-us>

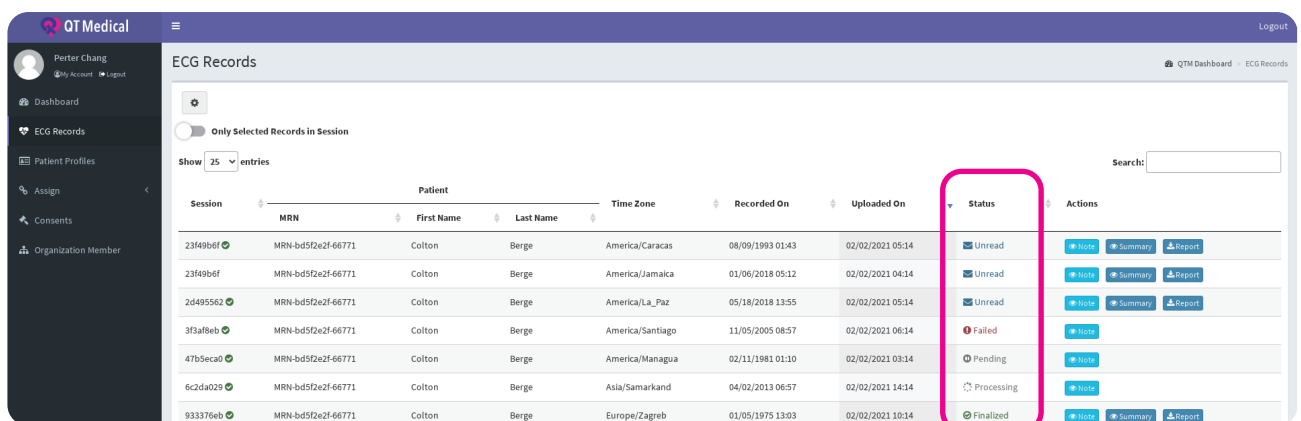

### 4.1.3 You can "Search", and turn on the option to show only Selected Records in a Session.

The screenshot shows the QT Medical interface. On the left is a sidebar with navigation links: Dashboard, ECG Records, Patient Profiles, Assign, Consents, and Organization Member. The main area is titled 'ECG Records'. At the top left of this area, there is a toggle switch labeled 'Only Selected Records in Session' which is currently turned on. A pink circle highlights this toggle, with the word 'Selected' written next to it. At the top right of the main area, there is a search bar. A pink circle highlights the search bar, with the word 'Search' written next to it. Below the search bar is a table with columns: Session, MRN, Patient (First Name, Last Name), Time Zone, Recorded On, Uploaded On, Status, and Actions. The table contains five rows of data. The first row has a status of 'Unread'. The second row has a status of 'Unread'. The third row has a status of 'Failed'. The fourth row has a status of 'Pending'. The fifth row has a status of 'Processing'.

Session	MRN	Patient	Time Zone	Recorded On	Uploaded On	Status	Actions
		First Name	Last Name				
2349b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread
3f3af8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending
6c2da029	MRN-bd5f2e2f-66771	Colton	Berge	Asia/Samarkand	04/02/2013 06:57	02/02/2021 14:14	Processing

### 4.1.4 Click on "Note", to view the Note in the popup window.

This screenshot is similar to the previous one, but with a pink circle highlighting the 'Note' button in the Actions column of the first row of the table. The 'Only Selected Records in Session' toggle is still turned on.

Session	MRN	Patient	Time Zone	Recorded On	Uploaded On	Status	Actions
		First Name	Last Name				
2349b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread
3f3af8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending
6c2da029	MRN-bd5f2e2f-66771	Colton	Berge	Asia/Samarkand	04/02/2013 06:57	02/02/2021 14:14	Processing

This screenshot shows the same ECG Records page, but with a popup window open. The popup window has a title bar that says 'Note for ECG ID: a78aa272-695f-4b46-92c0-2e88486a332f'. Inside the popup, there is a text box that says 'This is the correct record.' The background of the page is dimmed.



## 5. ECG Summary View

### 5.1 Measurements Adjustment (Physician Account ONLY)

5.1.1 Click on the **"ECG Records"**, you can see all uploaded ECG records from the dashboard.

QTM Medical

Perter Chang

Dashboard

**ECG Records**

Patient Profiles

Assign

Consents

Organization Member

ECG Records

Only Selected Records in Session

Show 25 entries

Search:

Session	MRN	First Name	Last Name	Time Zone	Recorded On	Uploaded On	Status	Actions
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread	<a href="#">Node</a> <a href="#">Summary</a> <a href="#">Report</a>
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Jamaica	01/06/2018 05:12	02/02/2021 04:14	Unread	<a href="#">Node</a> <a href="#">Summary</a> <a href="#">Report</a>
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread	<a href="#">Node</a> <a href="#">Summary</a> <a href="#">Report</a>
3f3a8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed	<a href="#">Node</a>
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending	<a href="#">Node</a>

Click on the **"Summary"**, you can see the summary view.

QTM Medical

Perter Chang

Dashboard

ECG Records

Patient Profiles

Assign

Consents

Organization Member

ECG Records

Only Selected Records in Session

Show 25 entries

Search:

Session	MRN	First Name	Last Name	Time Zone	Recorded On	Uploaded On	Status	Actions
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Caracas	08/09/1993 01:43	02/02/2021 05:14	Unread	<a href="#">Node</a> <a href="#">Summary</a> <a href="#">Report</a>
23f49b6f	MRN-bd5f2e2f-66771	Colton	Berge	America/Jamaica	01/06/2018 05:12	02/02/2021 04:14	Unread	<a href="#">Node</a> <a href="#">Summary</a> <a href="#">Report</a>
2d495562	MRN-bd5f2e2f-66771	Colton	Berge	America/La_Paz	05/18/2018 13:55	02/02/2021 05:14	Unread	<a href="#">Node</a> <a href="#">Summary</a> <a href="#">Report</a>
3f3a8eb	MRN-bd5f2e2f-66771	Colton	Berge	America/Santiago	11/05/2005 08:57	02/02/2021 06:14	Failed	<a href="#">Node</a>
47b5eca0	MRN-bd5f2e2f-66771	Colton	Berge	America/Managua	02/11/1981 01:10	02/02/2021 03:14	Pending	<a href="#">Node</a>

QTM

Summary View

MRN: MRN-bd5f2e2f-66771

Name: Colton Berge

Recorded: 2018-05-18 13:55:30 UTC-04:00

Uploaded: 2021-02-02 05:14:00 UTC-04:00

Gender: Male

Age: 25yrs

Date Of Birth: 07/05/1992 (mm/dd/yyyy)

Ven. Rate: 79 bpm

PR Interval: 171 ms

QRS duration: 105 ms

QT/QTc Interval: 371/428 ms

QTcf: -ms

P, QRS, T Axis: 70, 70, 55

Caliper

Chang: 2021-02-20 10:05:46 UTC+00:00

Normal sinus rhythm

Couplet atrial premature beat

Prolonged QT interval

Preliminary Interpretation: 2021-02-20 23:05:15 UTC+13:00

Normal sinus rhythm

Couplet atrial premature beat

Prolonged QT interval

\*\*Not Confirmed by Physician\*\*

Doctor's Comment

Doctor's Name

Normal sinus rhythm


Couplet atrial premature beat

Prolonged QT interval


Submit

Back to list


If you want to change the measurement, click the "numerical value".

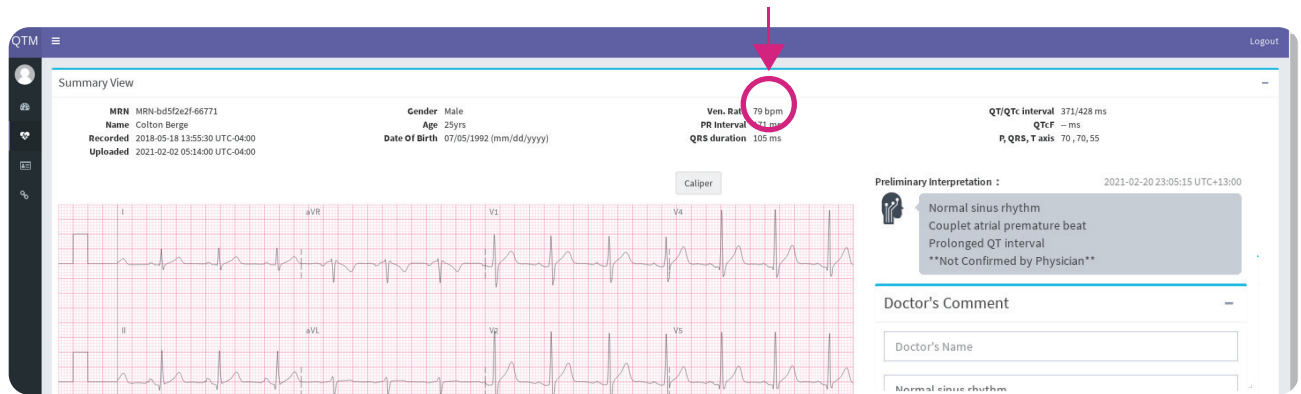

You will see the edit bar.

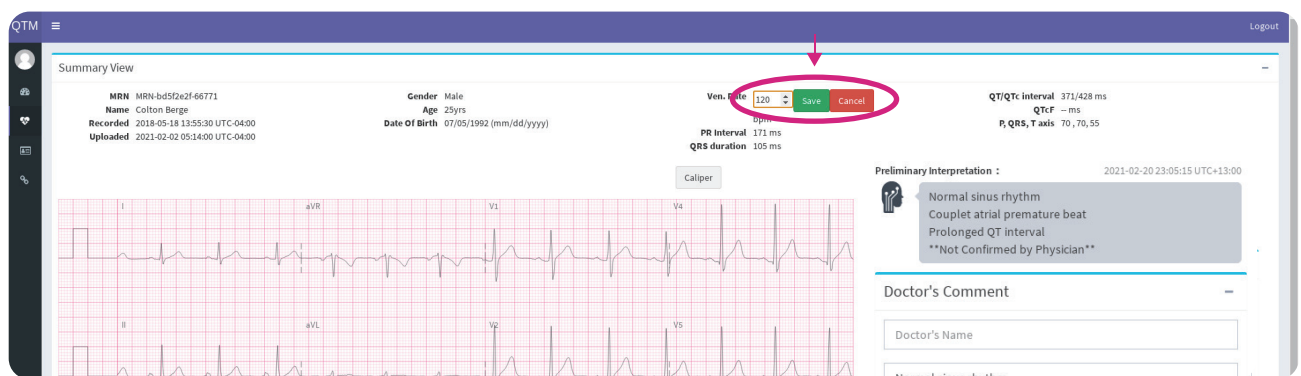

After editing, the numerical value will become bold.

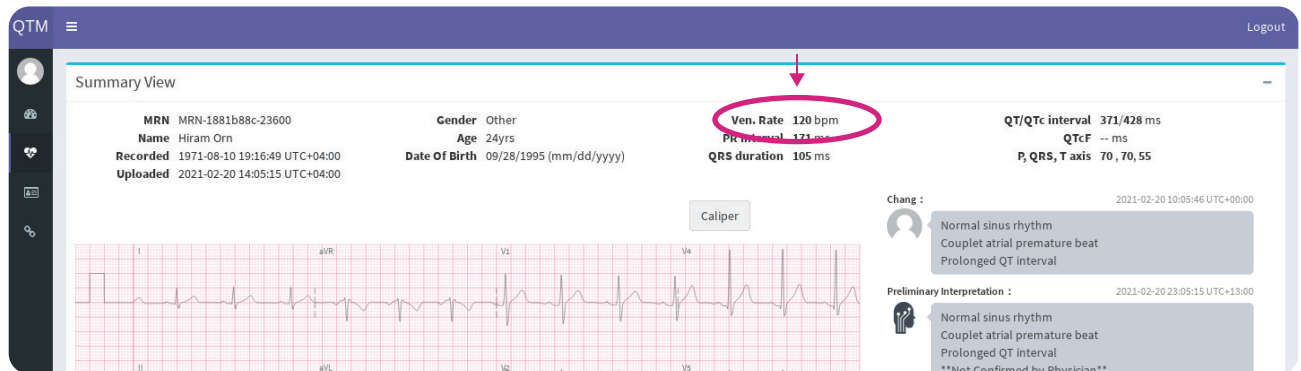

## 5.2 Caliper (Physician Account ONLY)

Select "**Caliper**" to show the movable green line and blue lines.

The screenshot shows the 'Summary View' of an ECG recording in the QTM application. The patient information includes MRN: 1881b88c-23600, Name: Hiram Orn, Gender: Other, Age: 24yrs, Date of Birth: 09/28/1995 (mm/dd/yyyy), Recorded: 2021-08-10 19:16:49 UTC+04:00, and Uploaded: 2021-02-20 14:05:15 UTC+04:00. The ECG parameters displayed are Ven. Rate: 120 bpm, PR Interval: 171 ms, QRS duration: 105 ms, QT/QTc interval: 371/428 ms, QTcF: -- ms, and P, QRS, T axis: 70, 70, 55. The ECG trace shows leads I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, and V6. A red circle highlights the 'Caliper' button in the top right corner of the ECG display area. The right sidebar contains a 'Chang' section with a timestamp of 2021-02-20 10:05:46 UTC+00:00, a 'Preliminary Interpretation' section with a timestamp of 2021-02-20 23:05:15 UTC+13:00, and a 'Doctor's Comment' section with a text area and 'Submit' and 'Back to list' buttons.


You can move the green and blue lines to measure the time interval in any part of the record.

The screenshot shows the 'Summary View' of an ECG recording in the QTM application. The patient information includes MRN: bd5f2e2f-66771, Name: Colton Berge, Gender: Male, Age: 25yrs, Date of Birth: 07/05/1992 (mm/dd/yyyy), Recorded: 2018-05-18 13:55:30 UTC-04:00, and Uploaded: 2021-02-02 05:14:00 UTC-04:00. The ECG parameters displayed are Ven. Rate: 120 bpm, PR Interval: 171 ms, QRS duration: 105 ms, QT/QTc interval: 371/428 ms, QTcF: -- ms, and P, QRS, T axis: 70, 70, 55. The ECG trace shows leads I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, and V6. A red circle highlights the 'Caliper' button in the top right corner of the ECG display area. The right sidebar contains a 'Chang' section with a timestamp of 2021-02-20 10:05:46 UTC+00:00, a 'Preliminary Interpretation' section with a timestamp of 2021-02-20 23:05:15 UTC+13:00, and a 'Doctor's Comment' section with a text area and 'Submit' and 'Back to list' buttons. The bottom of the ECG display shows a table of leads and their corresponding time intervals.

Lead	I	II	III	aVR	aVL	aVF	V1	V2	V3	V4	V5	V6	Rhythm	ms
Diff.	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	1000	3000	ms



The time interval will be measured.

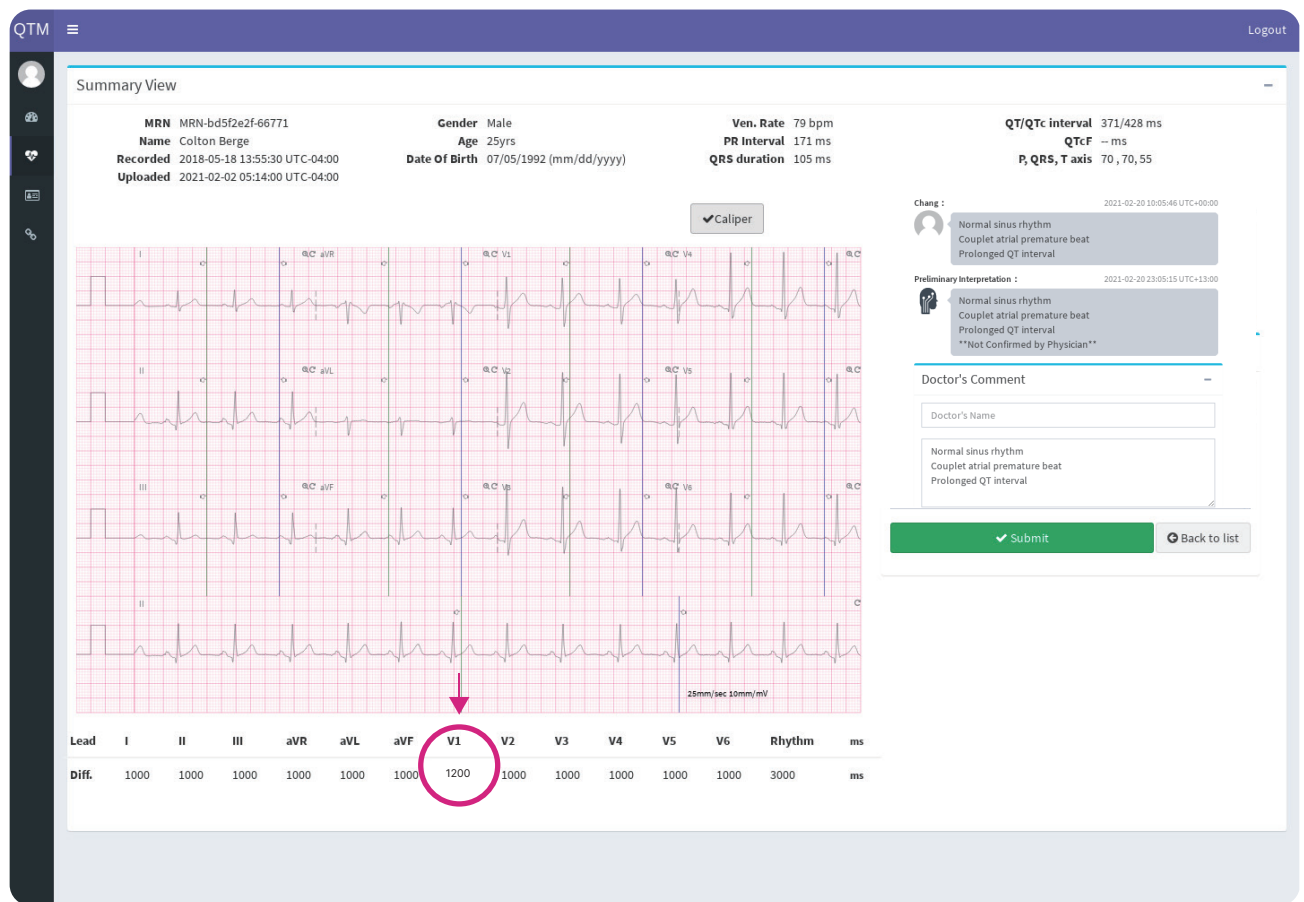

## 5.3 ECG Diagnosis (physician account ONLY)

### 5.3.1 Preliminary Interpretation is shown on the summary view.

The screenshot displays the 'Summary View' of an ECG report. At the top, patient information is shown: MRN (MRN-bd5f2e2f-66771), Name (Colton Berge), Gender (Male), Age (25yrs), Date Of Birth (07/05/1992), Ven. Rate (79 bpm), PR Interval (171 ms), QRS duration (105 ms), QT/QTc interval (371/428 ms), QTcF (ms), and P, QRS, T axis (70, 70, 55). The ECG traces are displayed in a 12-lead format (I, II, III, aVR, aVL, aVF, V1, V2, V3, V4, V5, V6). A 'Caliper' tool is visible above the traces. On the right, a 'Preliminary Interpretation' section is highlighted with a red box, containing the text: 'Normal sinus rhythm', 'Couplet atrial premature beat', 'Prolonged QT interval', and '\*\*Not Confirmed by Physician\*\*'. Below this is a 'Doctor's Comment' section with a text area containing the same text and a 'Submit' button.

### 5.3.2 Final Interpretation.

Physicians can make annotations in the **"Doctor's Comment"** section.

This screenshot is identical to the one above, but with the 'Doctor's Comment' section highlighted by a red box. The 'Preliminary Interpretation' section is no longer highlighted. The 'Doctor's Comment' section contains the same text as the preliminary interpretation: 'Normal sinus rhythm', 'Couplet atrial premature beat', 'Prolonged QT interval', and '\*\*Not Confirmed by Physician\*\*'. The 'Submit' button is visible at the bottom of the comment section.

Final interpretation is shown on the summary view.

QTM

Logout

Summary View

MRN: 1881b88c-23600

Name: Hiram Orr

Recorded: 1971-08-10 19:16:49 UTC+04:00

Uploaded: 2021-02-20 14:05:15 UTC+04:00

Gender: Other

Age: 24yrs

Date Of Birth: 09/28/1995 (mm/dd/yyyy)

Ven. Rate: 120 bpm

PR Interval: 171 ms


QRS duration: 105 ms

QT/QTc interval: 371/428 ms

QTcF: -- ms

P, QRS, T axis: 70, 70, 55

Caliper

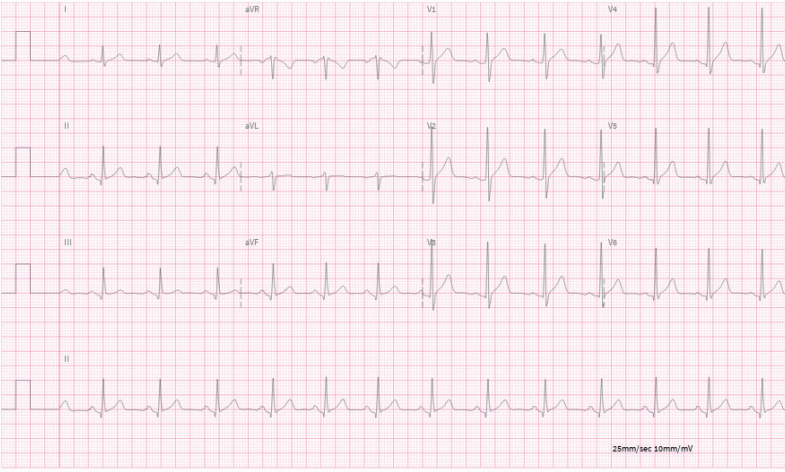

25mm/sec 10mm/mV

Chang : 2021-02-20 10:05:46 UTC+00:00

Normal sinus rhythm

Couplet atrial premature beat

Prolonged QT interval

Preliminary Interpretation : 2021-02-20 23:05:15 UTC+13:00

Normal sinus rhythm

Couplet atrial premature beat

Prolonged QT interval

\*\*Not Confirmed by Physician\*\*

Doctor's Comment

Doctor's Name

Normal sinus rhythm

Couplet atrial premature beat

Prolonged QT interval

Submit


Back to list

25

## 6. Patient Profile Management


### 6.1 Overview

6.1.1 Click **"Patient profile"**, you can see all uploaded patient profile from the dashboard.

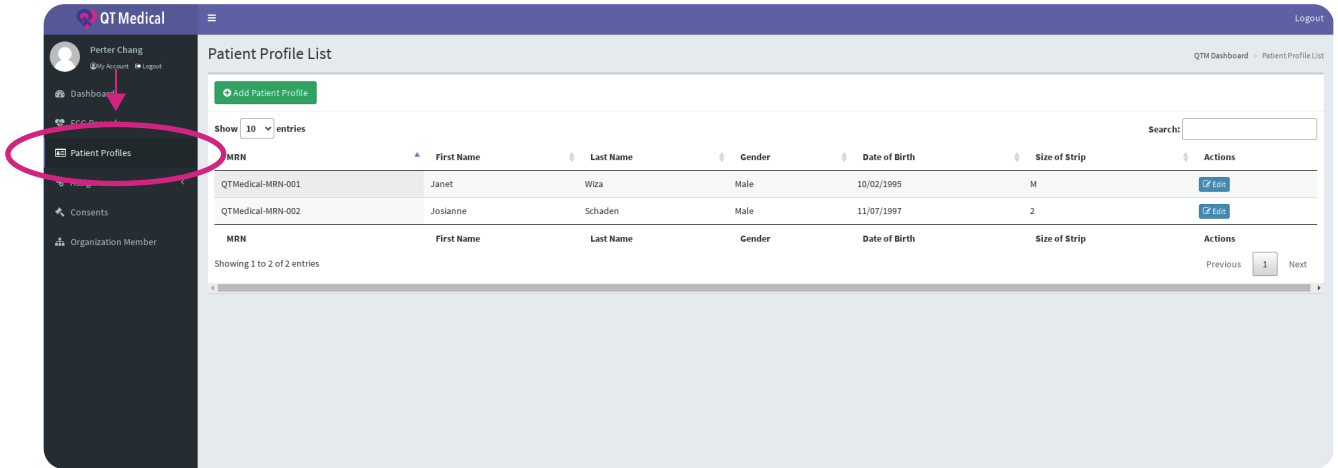

#### 6.1.2. Add Patient Profile

Click **"Add Patient Profile"** button.

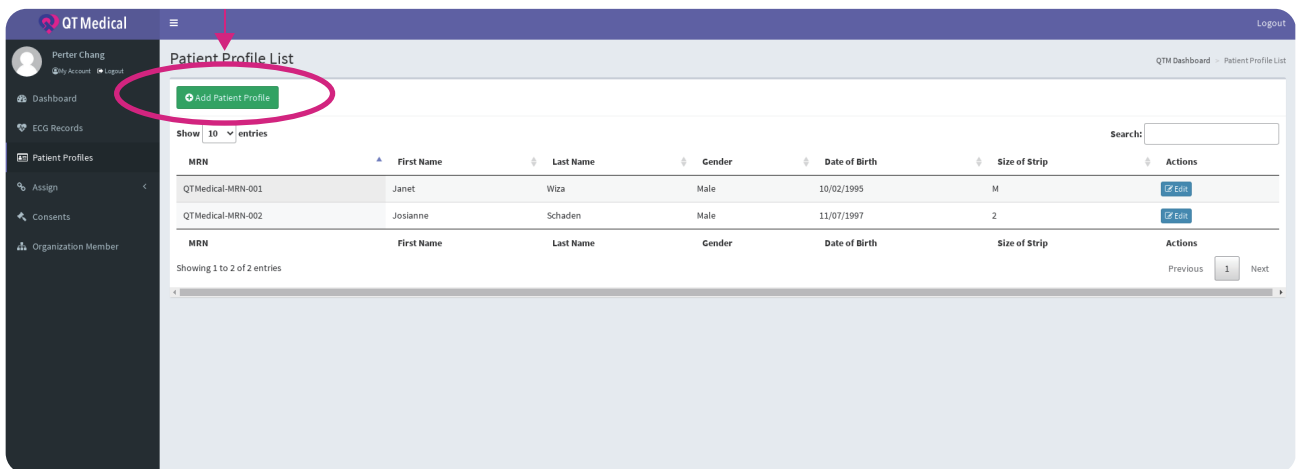

Fill in the patient profile and Click **"Create"**.

QT Medical

Perter Chang  
My Account Logout

Dashboard  
ECG Records  
Patient Profiles  
Assign  
Consents  
Organization Member

### Create Patient Profile

QTM Dashboard Patient Profile List Create Patient Profile

\* MRN  
MRN

\* First Name  
First Name

\* Last Name  
Last Name

\* Gender  
Please choose your gender

\* Date of Birthday  
mm/dd/yyyy

\* Size of Strip  
Please choose your size

Create

The patient profile is created.

QT Medical

Perter Chang  
My Account Logout

Dashboard  
ECG Records  
Patient Profiles  
Assign  
Consents  
Organization Member

### Patient Profile List

QTM Dashboard Patient Profile List

Add Patient Profile

Show 10 entries

Search:

MRN	First Name	Last Name	Gender	Date of Birth	Size of Strip	Actions
QT Medical-MRN-001	Janet	Wiza	Male	10/02/1995	M	Edit
QT Medical-MRN-002	Josianne	Schaden	Male	11/07/1997	2	Edit
QT Medical-MRN-003	Jimmy	Lu	Male	09/29/1990	XL	Edit


Showing 1 to 3 of 3 entries

Previous 1 Next



### 6.1.3 Edit Patient Profile


Click the **"Edit"** button on the right.

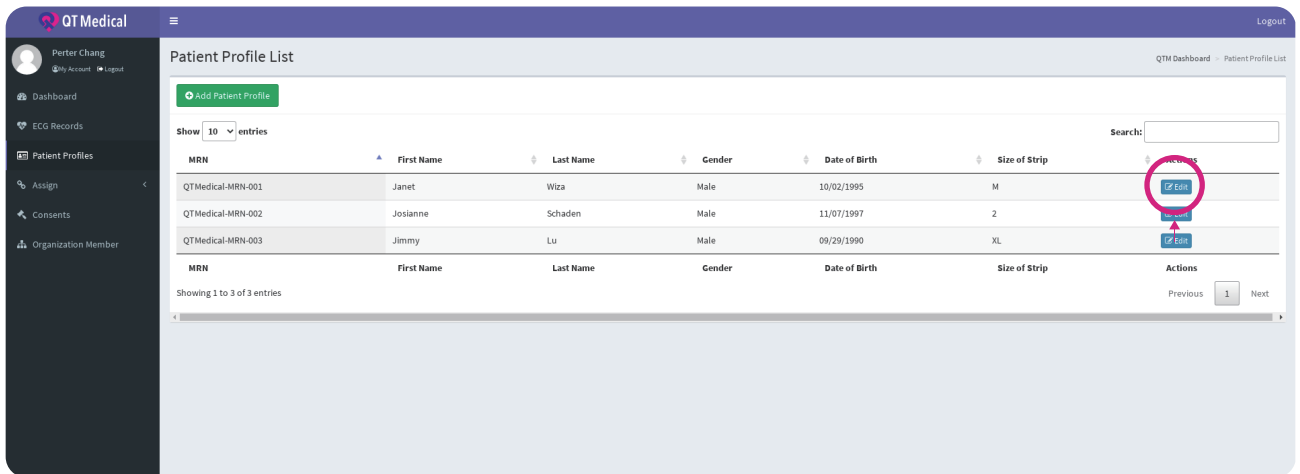

The screenshot shows the 'Patient Profile List' interface. On the left is a sidebar with navigation options: Dashboard, ECG Records, Patient Profiles, Assign, Consents, and Organization Member. The main area displays a table of patient profiles. The first entry is highlighted, and its 'Edit' button is circled in red.

MRN	First Name	Last Name	Gender	Date of Birth	Size of Strip	Actions
QTMEDICAL-MRN-001	Janet	Wiza	Male	10/02/1995	M	<a href="#">Edit</a>
QTMEDICAL-MRN-002	Josianne	Schaden	Male	11/07/1997	2	<a href="#">Edit</a>
QTMEDICAL-MRN-003	Jimmy	Lu	Male	09/29/1990	XL	<a href="#">Edit</a>

Start editing and click **"Update"** to save.

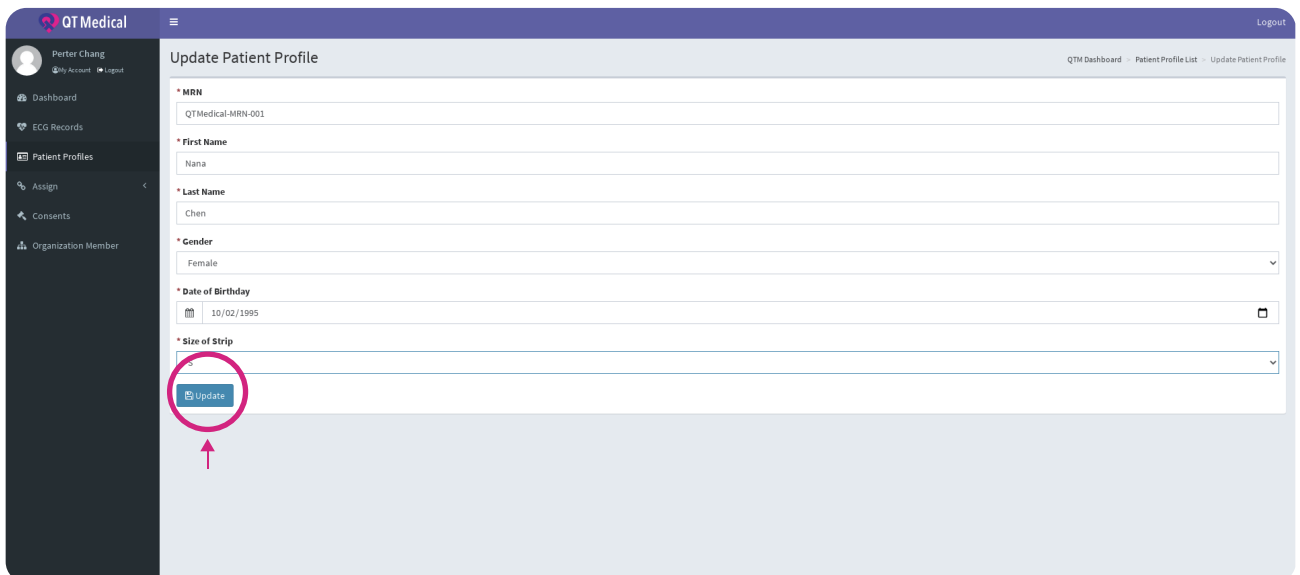

The screenshot shows the 'Update Patient Profile' interface. The form contains fields for MRN, First Name, Last Name, Gender, Date of Birth, and Size of Strip. The 'Update' button is circled in red.

\* MRN  
QTMEDICAL-MRN-001

\* First Name  
Nana

\* Last Name  
Chen

\* Gender  
Female

\* Date of Birth  
10/02/1995

\* Size of Strip  
S

[Update](#)



## 7. Assign

### 7.1 Patient Profile to ECG

Click **"Assign"**, select **"Assign ECG to Patient"**. You will see the list by session.

QT Medical

Perter Chang

Dashboard

ECG Records

Patient Profiles

Assign

Assign ECG to Patient

Consents

Organization Member

Assign ECG to Patient

QTM Dashboard > Assign > Assign ECG to Patient

Show 10 entries

Search:

Session	Patient	Time Zone	Session Recording Started	Operator	Notes	Actions
564e51a0	Not Specified	Antarctica/Davis	04/01/1995 21:25	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
1c95a720	Lavern Watsica	America/Argentina/Buenos_Aires	02/07/1993 21:40	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	America/Kentucky/Monticello	10/18/2004 09:11	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	Indian/Cocos	10/18/2004 19:41	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>

Showing 1 to 4 of 4 entries

Previous 1 Next

ECG Records created by Guest mode will be displayed as "Not Specified" and will need to be reassigned to the correct patient.

QT Medical

Perter Chang

Dashboard

ECG Records

Patient Profiles

Assign

Assign ECG to Patient

Consents

Organization Member

Assign ECG to Patient

QTM Dashboard > Assign > Assign ECG to Patient

Show 10 entries

Search:

Session	Patient	Time Zone	Session Recording Started	Operator	Notes	Actions
564e51a0	Not Specified	Antarctica/Davis	04/01/1995 21:25	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
1c95a720	Lavern Watsica	America/Argentina/Buenos_Aires	02/07/1993 21:40	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	America/Kentucky/Monticello	10/18/2004 09:11	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	Indian/Cocos	10/18/2004 19:41	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>

Showing 1 to 4 of 4 entries

Previous 1 Next

Click **"Reassign"**, select the patient profile list.

When assigned successfully, you will see the success message popup.

Assign ECG to Patient

Show 10 entries

Search:

Session	Patient	Time Zone	Session Recording Started	Operator	Notes	Actions
564e51a0	Not Specified	Antarctica/Davis	04/01/1995 21:25	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
1c95a720	Lavern Watsica	America/Argentina/Buenos_Aires	02/07/1993 21:40	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	America/Kentucky/Monticello	10/18/2004 09:11	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	Indian/Cocos	10/18/2004 19:41	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>

Showing 1 to 4 of 4 entries

Previous 1 Next

Patient List for Session ID: 564e51a0

Show 10 entries

Search:

MRN	First Name	Last Name	Actions
MRN-5a108746-71232	Lavern	Watsica	<a href="#">Select</a>

Showing 1 to 1 of 1 entries

Previous 1 Next

And you can click the **"Detail"**, you can see details of Note.

Assign ECG to Patient

Show 10 entries

Search:

Session	Patient	Time Zone	Session Recording Started	Operator	Notes	Actions
564e51a0	Not Specified	Antarctica/Davis	04/01/1995 21:25	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
1c95a720	Lavern Watsica	America/Argentina/Buenos_Aires	02/07/1993 21:40	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	America/Kentucky/Monticello	10/18/2004 09:11	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>
f109a009	Lavern Watsica	Indian/Cocos	10/18/2004 19:41	Anthony Xu	<a href="#">Detail</a>	<a href="#">Reassign</a>

Showing 1 to 4 of 4 entries

Previous 1 Next

Notes for ECG Session ID: f109a009

ECG #0 Note

The patient moves during recording.

ECG #1 Note

This is the correct record.

# Customer Service Support

For customer service support, please contact:



## QT Medical

1370 Valley Vista Dr., Suite 266  
Diamond Bar, CA 91765 U.S.A.

TEL : +1 855-722-6006

FAX : +1 310-755-3108

[service@qtmedical.com](mailto:service@qtmedical.com)

[www.qtmedical.com](http://www.qtmedical.com)



MedNet EC-REP GmbH  
Borkstrasse 10, 48163 Muenster, Germany

UK Responsible Person:  
MediMap Ltd, VAT Number 860 8763 93, registered at 2,  
The Drift, Thurston, Suffolk, IP31 3RT United Kingdom